



BYTE AV GUMMIÖVERDRAG - KÖRBÅGE BAMBINO 3

Monteringsanvisning för:

Byte av gummiöverdrag på Körbåge Bambino 3 (art.nr. 4233000)

För beställning av nytt gummiöverdrag använd art.nr. 4333028

Verktyg som behövs:

Insexnyckel 4 mm

Tunn skruvmejsel



1)
Dra tillbaka skum-greppet på toppen av körbågen tills skruven blir synlig.

Lossa skruven med M4 insex nyckel.



2)
Dra tillbaka skum-greppet på botten av körbågen.
Bänd bort plast kåpan med en smal skruvmejsel eller liknande verktyg.

Upprepa på andra sidan.



3)
Lossa på de 2 skruvarna med M4 insex nyckel.

Upprepa på andra sidan.



4)
Dra ut körbågen åt sidan.
Drag av skum-greppet från körbågen.
Trä på det nya skum-greppet på körbågen.

Upprepa på andra sidan.



5)
Tryck fast körbågen på körbågsfästet.
Skruva tillbaka de två skruvarna.

Upprepa på andra sidan.



6)
Pressa tillbaka plast kåpan.
Trä det nya skum-greppet så att det kommer i kontakt med kåpan.

Upprepa på andra sidan.



7)

Drag åt skruven på toppen av körbågen.
För skum-greppet över skruven så att det inte
blir något gap mellan de två skum-greppen.

REPLACE FOAM GRIP ON PUSHBAR BAMBINO 3



Assembly instructions for:
Replacing the foam grip on Pushbar Bambino 3 (art.no. 4233000)

To order new foam grip use art.no. 4333028

Tools aquired:
Phillips screwdriver - small
Allen key 4 mm



1)
Pull back the foam at the top of the push bar until a screw is visible.

Unscrew the screw using a M4 allen key.



2)
Pull back the foam from the bottom of the handle bar.
Pry of the plastic cap using a thin flat screwdriver or similar tool.

Repeat on other side.



3)
Unscrew the 2 screws using a M4 allen key.

Repeat on other side



4)
Pull out the handle bar sideways.
Pull of the foam grip from the handle bar.
Insert new foam onto the handle bar

Repeat on other side



5)
Push the handle bar into the handle bar holder.
Screw back the 2 screws

Repeat on other side.



6)
Push the plastic cover back on.
Push the new foam grip so that it makes contact with the plastic cover.

Repeat on other side



7)
Tighten the screw at the top of the handlebar.
Push the foam grips over the screw making sure there is no gap between the two foam grips.