



### SITTBREDDSDAPTER BAMBINO 3



Monteringsanvisning för:  
Sittbreddsadapter Bambino 3  
art.nr. 4235021, 4235022

Verktyg som behövs:  
Inga

OBS! Eventuell förhöjningsdel B3 (4235012 / 4235013) måste demonteras innan montering av Sittbreddsadaptern och återmonteras efter att Sittbreddsadaptern har blivit monterad.

Med sittbreddsadapter begränsas ryggvinklingen framåt till max 16mm längd på ställskruv.



**1)** Trä ner aluprofilen (2) på sidoskyddet i hålet i botten på sittbreddsadaptern (1).



**2)** För upp adaptern längs profilen, tills den ligger över sidoskyddet. Pressa på övre och bakkanten av adaptern tills den snäpper fast över kanterna på sidoskyddet.



## SEAT WIDTH ADAPTER BAMBINO 3



Assembly instructions for:  
Seat width adapter Bambino 3  
art.nr. 4235021, 4235022

No tools needed

NOTE! Any extension part B3 (4235012/4235013) must be removed before mounting the seat width adapter and be remounted after the seat width adapter has been mounted.

The seat width adapter limits the back angle to 16mm forward on the angle adjustment screws.



**1)** Thread the aluminum profile (2) on the side guard into the hole in the bottom of the seat width adapter (1).



**2)** Slide the adapter along the profile until it covers the side guard. Press on the top and back edge of the adapter until it snaps in place over the edges of the side guard.