


keeps you moving


SIDOSKYDD MICRO 3

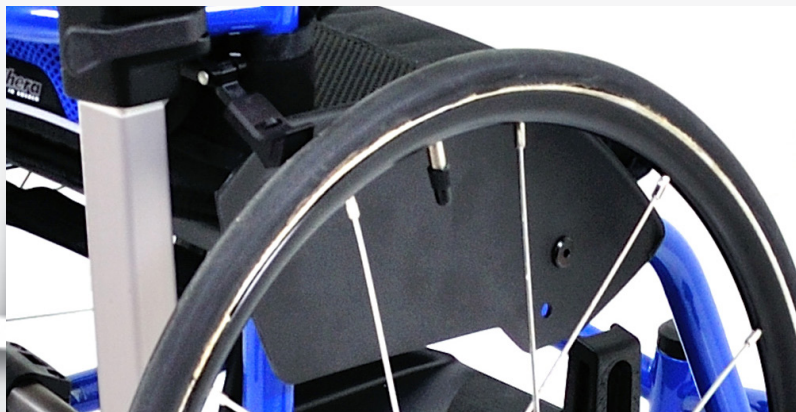

Monteringsanvisning för: Sidoskydd Micro 3

art.nr. 4400050 Sidoskydd M3 kpl med fäste par
art.nr. 4400053 Sidoskydd M3 kpl med fäste vänster
art.nr. 4400054 Sidoskydd M3 kpl med fäste höger

Verktyg som behövs:
Insexnyckel 3 mm

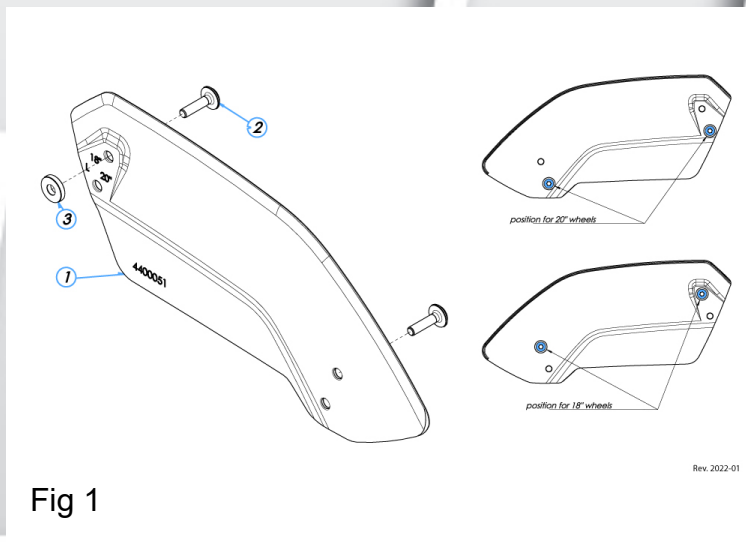

POS

1. 4400051 Sidoskydd Micro 3 vänster
4400052 Sidoskydd Micro 3 höger
2. 6004520 Bult MELC6S M5X20 FZSV
3. 4600042 Distans breddning 3 mm

Montering

Sidoskydd (vänster resp. höger) passar båda längder av chassin samt olika dimension på drivhjul, 18" och 20". På sidoskydd finns markering för vilka hål som ska användas för respektive hjulstorlek samt vänster respektive höger sida (L resp R). Se Fig 2

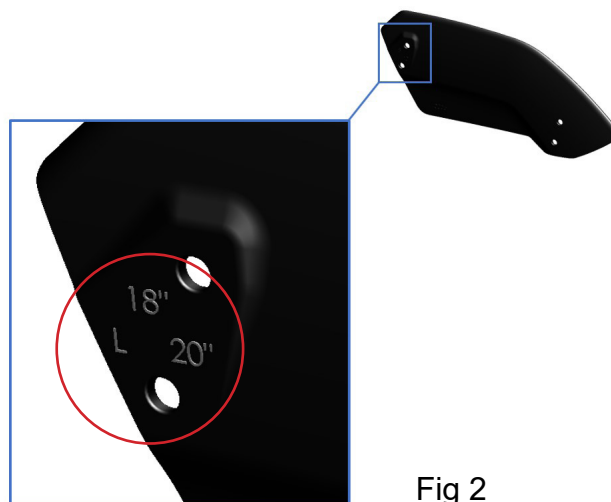




Fig 3

1) Placera sideskyddet (höger sida) mot chassit enligt bilden och fäst den främre skruven löst. Fig. 3 (monterad för 18" drivhjul)

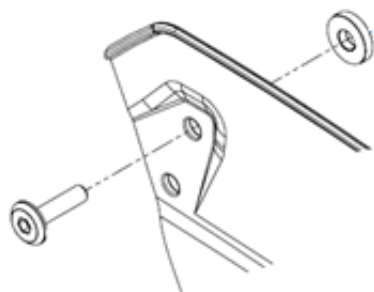


Fig 4

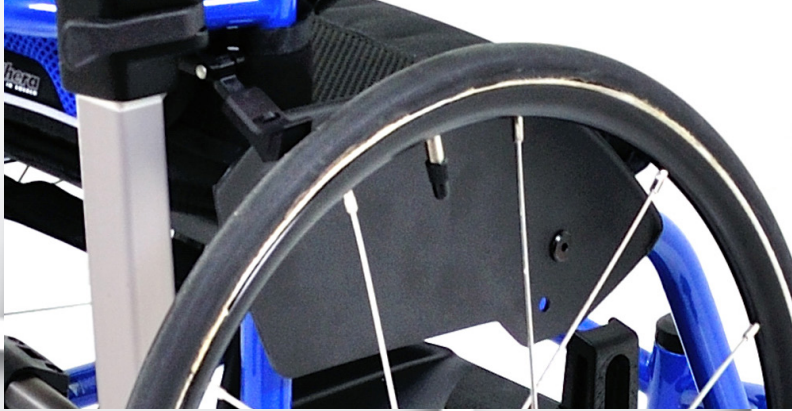


Fig 5

2) Lägg brickan mellan chassit och sideskyddet vid det bakre hålen och fäst sideskydd och bricka med en skruv mot chassit Fig 4. Drag åt skruvarna med insexnyckel 3 mm med ett lätt tryck Fig 5.

keeps you moving


SIDE GUARD MICRO 3



Assembly instruction for: Side guard Micro 3

art.no. 4400050 Sideguard M3 cpl w. attachment pair
art.no. 4400053 Sideguard M3 cpl w attachment left
art.no. 4400054 Sideguard M3 cpl w attachment right

Tools needed:
Allen key 3 mm

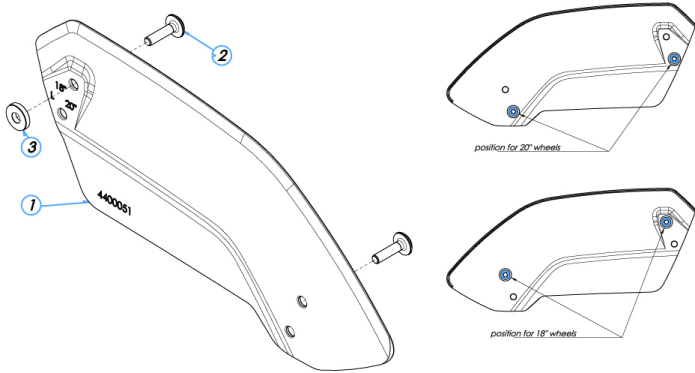

Rev. 2022-01

Fig 1

POS

1. 4400051 Sideguard M3 left
4400052 Sideguard M3 right
2. 6004520 Bolt MELC6S M5X20 FZSV
3. 4600042 Spacer widening high brake 3 mm

Mounting

The side guard fits (left and right) both chassis length and also the two rear wheel dimensions 18" and 20". There is marking on the sideguard for which hole should be used for each rear wheel size and also if it is left or right side (L or R). See Fig 2

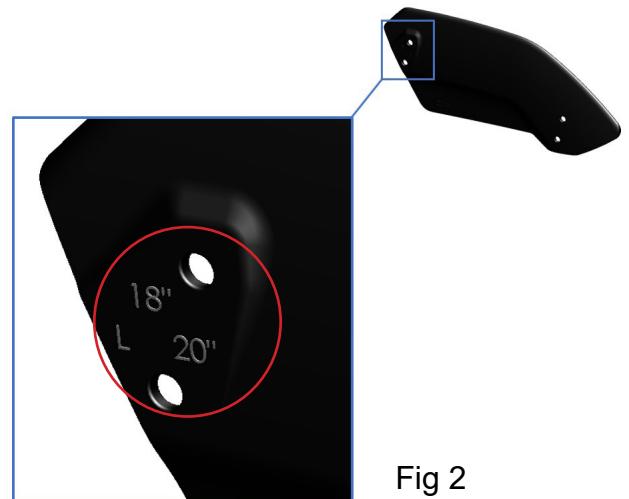


Fig 2

panthera®

keeps you moving



Fig 3

1) Place the side guard (right side) on the chassis as in the picture and fasten the front screw slightly. Fig. 3 (mounted for 18" rear wheels)

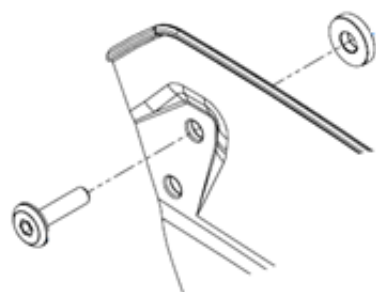


Fig 4



Fig 5

2) Place the washer between the chassis and the side guard at the rear hole and attach the side guard with the washer and screw to the chassis Fig 4. Tighten the screws with allen key 3 mm with a light push Fig 5.