

SEAT BAG



SEAT MOUNTED art.no. 9900212



CUSHION MOUNTED art.no. 9900213

The seat bag are available in two versions, seat mounted or cushion mounted.

- The **seat mounted** seat bag is attached directly to the velcro at the seat, and remain mounted to the wheelchair.
 - The **cushion mounted** seat bag is attached to the cushion, and remain mounted to the cushion when lifted out of the wheelchair. The cushion mounted bag is delivered together with a self-adhesive velcro tape that you attached to the underside of the cushion.(This stage is no longer necessary when the S3 cushion now has a premounted velcro) The velcro of the seat bag is then attached to this.
- The seat bag's loose end is attached to the chassis tubes with magnets.



Mounting of **seat mounted** bag to the velcro of the seat.



Mounting of the enclosed velcro tape to the underside of the cushion on **cushion mounted** bag.
(This stage is no longer necessary when the S3 cushion now has a premounted velcro)
Attache the cushion mounted bag's velcro to this tape.



Behind the zipper is a large compartment for storage of e.g. keys etcetera.



If you lift the three flaps locked with magnets you find more storage for e.g. cellphone, wallet etcetera



Use the seat bag's magnets to attached the bag to the chassis tube when driving the wheelchair.