
# SmartDrive MX2+ | PushTracker E3

## User's Manual

Informational videos available at  
[www.permobilsmartdrive.com](http://www.permobilsmartdrive.com)

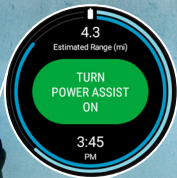E3SM2-USM Rev. A

permobil

## Table of Contents

1. Introduction . . . . .	2
Intended Use of SmartDrive MX2+ App . . . . .	2
Warnings and Safety Precautions . . . . .	2
Electromagnetic Interference (EMI) . . . . .	4
Bluetooth Wireless Communication . . . . .	4
2. Using Your PushTracker E3 . . . . .	4
Attaching Your PushTracker E3 . . . . .	4
Initial Setup . . . . .	5
SmartDrive Pairing . . . . .	6
Power Assist Activated . . . . .	6
3. Companion App . . . . .	7
PushTracker App. . . . .	7
4. Maintenance . . . . .	7
Inspection . . . . .	7
Updates and Other Maintenance . . . . .	8
Troubleshooting . . . . .	8
5. Warranty . . . . .	9
Manufacturer's Contact Information . . . . .	9
6. European Commission (EC) Authorized Representative . . . . .	9

## 1. Introduction

### Intended Use of SmartDrive MX2+ App

The SmartDrive MX2+ software application (app) is intended to be used on the PushTracker E3 (TicWatch E3) to operate the SmartDrive Wheelchair Power Assist device.

Refer to the labeling and documentation of other components or product for their intended use, instruction, wireless, EMI and all other specification / information.

### Warnings and Safety Precautions

All warnings and safety precautions issued by one's wheelchair manufacturer must be heeded in addition to those stated in this and any other provided manuals / instructions / guides, to include that provided within the SmartDrive MX2+ app. Failure to do so could result in serious injury or death.

Use of the SmartDrive MX2+ app or the SmartDrive by anyone who has not received proper instruction from this manual, the SmartDrive User's Manual and any other provided manuals / instructions / guides may result in serious injury or death.

Use of the SmartDrive MX2+ app is to be ONLY on the PushTracker E3 (TicWatch E3) as made available by Permobil [SmartDrive] / Max Mobility.

Always inspect your PushTracker E3 for missing parts / damage prior to each use.

Make sure your PushTracker E3 is charged before use.

When crossing major roads, intersection, railway crossings or highways as well as when you drive steep, long slopes you should always consider having somebody accompany you in the interest of your safety.

The PushTracker E3 should ONLY be worn by the user sitting in the wheelchair with the SmartDrive attached.

Always turn OFF the power assist [via SmartDrive MX2+ app] when parked to prevent

*(continued on next page...)*

accidental activation.

Always turn OFF the power assist [via the SmartDrive MX2+ app] before taking the PushTracker E3 off of your wrist.

Tapping force does not need to be excessive or "hard". Gentle, stiff wrist taps at a reasonable speed [i.e. one half (0.5) second between taps] is sufficient. Refer to the SmartDrive User's Manual for complete instruction.

Always use caution when transferring in or out of the wheelchair. Every precaution should be taken to reduce the transfer distance and be certain the wheel locks are engaged to prevent the wheels from moving. Also always make sure the power assist is OFF [via the SmartDrive MX2+ app] when transferring to and from your chair.

The PushTracker E3 has passed industry climate testing, and is considered swim-ready [up to 50 m]. Refer to the other provided manuals / instructions / guides for complete detail regarding its water resistance performance.

Ensure your device is running the latest versions.

Access to the Google Play Store to install the SmartDrive MX2+ [and any other app] on the PushTracker E3 is restricted. Do not uninstall or remove any apps / software and refer to the Troubleshooting section(s) for more information if done accidentally or against this instruction.

#### NOTES:

*Access to the Google Play Store to install the SmartDrive MX2+ [and any other app] on the PushTracker E3 is restricted. Separate policies, terms and conditions apply if access is gained against this instruction.*

*Figures and diagrams in this manual may vary slightly from your device.*

*All product [including components], requirements, policies, terms and conditions, to include the contents of this and any other provided manual / instruction / guide, are subject to change without prior notice. For current product instruction and information refer to the official website.*

*An electronic version of this User's Manual along with informational videos are available on our website [www.pernobilsmartdrive.com] for people with visual, reading or cognitive disabilities.*

*The Serial Number of your PushTracker E3 can be found in the Settings app.*

*It is recommended that you save this User's Manual and record the Serial Number below for reference.*

### Device Serial Number:

## Electromagnetic Interference (EMI)

Refer to the SmartDrive User's Manual for EMI specifications.

## Bluetooth Wireless Communication

 Bluetooth wireless communication [2.4 GHz frequency band] is used by the PushTracker E3 to communicate to the SmartDrive when to activate/deactivate the power assist, send and receive signals from a smartphone, etc. The PushTracker E3 must be within 25 ft (8 meters) of the SmartDrive / smartphone to connect and communicate. The maximum output power of this technology is 10 mW, and the operation of this device in close proximity to other wireless devices [i.e. electromagnetic security systems, Bluetooth headsets, etc.] could cause the device to lose connection. Power assist is always deactivated whenever the motor is driving and there is a loss of connection; the SmartDrive will beep and PushTracker E3 will also give indication when this happens. Worst case coexistence testing was performed on the device to ensure its safety and effectiveness. Please contact Permobil [SmartDrive] / Max Mobility immediately for troubleshooting if continuous connection issues are experienced with the device.

## 2. Using Your PushTracker E3

### Attaching Your PushTracker E3

The PushTracker E3 is designed to comfortably and securely attach to the top part of wrists of all sizes. It has an embedded accelerometer to recognize when you push, when you want to activate the power assist and also when you tap your hand against the handrim to deactivate the

power assist. Additionally, it is equipped with an AMOLED screen and vibration motor to indicate different information, states and activations / deactivations that occur during everyday use. The band on the PushTracker E3 is designed for most users; but if you have difficulty securing the buckle / clasp, it doesn't fit right, or you wish to have one of a different color / material, the band can be easily replaced with essentially any 0.87 in (22 mm) wide off-the-shelf watch band of your choosing.

**It is recommended that your PushTracker be worn snug on your wrist to get the most reliable recognition of your hand movement for SmartDrive operation, propulsion activity tracking, etc. Ability to operate / maneuverer the wheelchair can be affected if the PushTracker, watches, jewellery, etc. are worn loosely or positioned such that they can become loose and interfere with handrim operation.**

## Initial Setup

After powering on, follow the prompts closely on the PushTracker E3. If any serious difficulties are experienced during set-up, go to the Permobil [SmartDrive] / Max Mobility website for more information and support.

Manual setting of the time, time zone and preferred format is required, and this is done in the separate Settings app. After opening that app, enter the System menu to then access the "Date & time" submenu. To manually set these, toggle OFF the "Automatic" options then configure each to the correct and / or preferred values.

Upon successful completion, open the SmartDrive MX2+ app on the PushTracker E3, either from the app list or from the button on the selectable SmartDrive MX2+ watchface.

SmartDrive MX2+

## SmartDrive Pairing

Initial pairing of the PushTracker E3 to a SmartDrive is required, and this is done in the SmartDrive MX2+ app. If the SmartDrive MX2+ app has not been previously paired to a SmartDrive, turning power assist ON will initiate the pairing. If the SmartDrive MX2+ app has been previously paired to a SmartDrive, pairing can be done in the Settings menu on the app.

## Power Assist Activated


After successful pairing, power assist is turned ON and the SmartDrive MX2+ app is connected to the SmartDrive, the Indication will flash blue. When the SmartDrive motor is driving, the Indication will be solid blue. The Indication will flash red when the SmartDrive MX2+ App is trying to connect or has lost wireless connection to the SmartDrive. Whenever there is a change of state to the SmartDrive, you will feel vibration as additional indication from your PushTracker E3.

When power assist is ON and the SmartDrive motor is driving, the SmartDrive MX2+ app will display your current speed. The Indication will light up with various colors to notify you of the SmartDrive's current state.

Additional detail / information that is helpful in explaining different features of the app and the SmartDrive is provided throughout the SmartDrive MX2+ app [by clicking (i)].

Refer to the SmartDrive User's Manual for its operation information and instruction.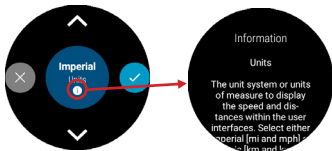

### 3. Companion App

#### PushTracker App

For the best experience with your SmartDrive and PushTracker E3, use the PushTracker app on your PushTracker E3. This will allow you to track your propulsion activity progress and more.

PushTracker

### 4. Maintenance

#### Inspection

Always inspect the PushTracker E3 for missing parts / damage prior to each use. Cease use and contact Permobil [SmartDrive] / Max Mobility or your mobility equipment dealer if you notice any of the following:

- Broken or cracked plastic
- Broken buckle or ripped bands
- Damaged display

## Updates and Other Maintenance

Check regularly to ensure your device is running the latest versions, as features are added and improvements are made.

## Troubleshooting

Contact Permobil [SmartDrive] / Max Mobility or your mobility equipment dealer for any other issues that you experience with the device or app. It is strongly recommended to update the versions on your device to the latest version prior to any troubleshooting as your issue may have been resolved in later releases.

**Li-ion**

Do not throw away the PushTracker E3. Please properly dispose of the battery by recycling at the end of its life or send it back to Permobil [SmartDrive] / Max Mobility.

## 5. Warranty

Permobil [SmartDrive] / Max Mobility offers a 1 year warranty against manufacturing defects. Permobil [SmartDrive] / Max Mobility's obligation under this warranty shall be limited to repairing or providing replacement of part(s), which are found to be defective. Any part(s) of the system replaced by us at our discretion shall be with a functionally operative part.

This warranty may be voided if: 1) the product is not used according to instructions given in this instruction manual, 2) defects were caused by improper use as determined by the company personnel, 3) modification or alteration of any nature is made in the electrical circuitry or physical construction, or 4) defects are due to cause beyond control like lightning, abnormal voltage or while in transit at the purchaser's place of business.

### Manufacturer's Contact Information

Manufacturer:	Max Mobility, LLC
Address:	300 Duke Drive, Lebanon, TN 37090 USA
Phone:	(615) 953-5350 / Toll-Free (800) 637-2980
Fax:	(888) 411-9027
Website:	<a href="http://www.permobilsmartdrive.com">www.permobilsmartdrive.com</a> / <a href="http://www.pushtracker.com">www.pushtracker.com</a>
Email:	<a href="mailto:support.smartdrive@permobil.com">support.smartdrive@permobil.com</a>

## 6. European Commission (EC) Authorized Representative

Advena Ltd.  
Tower Bus. Ctr., 2nd Floor  
Tower Street  
Swatar BKR 4013  
Malta

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