



CRUTCH HOLDER S3

Manual for:
Crutch holder S3
art.no. 9900301



[Link to video](#)

ATTACH THE CRUTCH ON THE WHEELCHAIR



1) Swing up the upper attachment



2) Push it on to the backrest bow.



3) Move the crutch's lower attachment towards the chassis until the magnet attaches to the metal tube.

REMOVE THE CRUTCH FROM THE WHEELCHAIR



1) Push the handle of the crutch until the magnetic attachment releases from the chassis.



2) Push the handle sideways until the attachment releases from the backrest bow..



3) Lift the crutch away from the wheelchair.