


## Onearmdrive

### User manual for onearmdrive with quick release.


Onearmdrive is an accessory for people with reduced or no function in one arm. It makes it possible to propel the wheelchair with just one arm, when both pushrims are mounted on one side. The smaller pushrim drives the opposite wheel and larger the closest. By gripping both pushrims with one hand and pushing forward the chair will go straight forward.

#### 1. Mounting of onearmdrive with quick release.

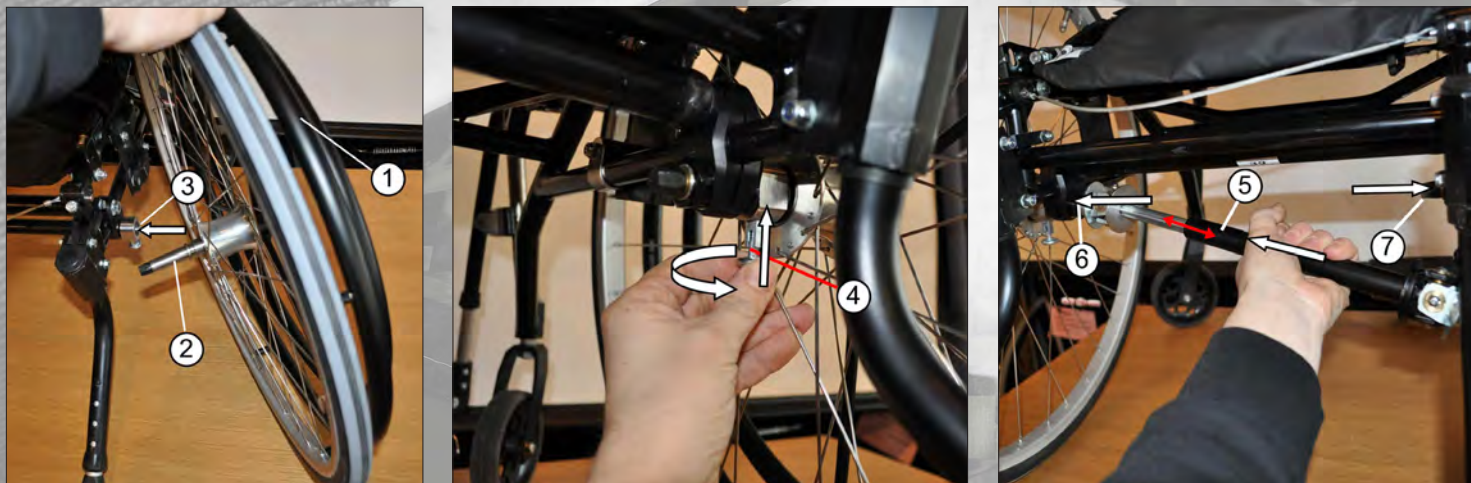

- 1) Mount the rearwheel with pushrims (1) on the side desired of the user.
- 2) Enter the rearwheel axle (2) into the attachment (3) until it stop.
- 3) Pull the locking pin (4) and check that the rearwheel axle (2) has enter fully.
- 4) Turn the locking pin (4) and release it so it locks in the slot in the rearwheel axle (2).
- 5) Check that the rearwheel (1) is secured by pulling it straight out.
- 6) Repeat the procedure on the other wheel.
- 7) Fit the driveshaft (5) end to one rearwheel.
- 8) Press the springy shaft (5) to fit the other end to the other rearwheel (7).
- 9) Check the function of the driveshaft by pulling the smaller pushrim (1) and note if it drives the wheel.

#### 2. Disassembly of onearmdrive with quick release.

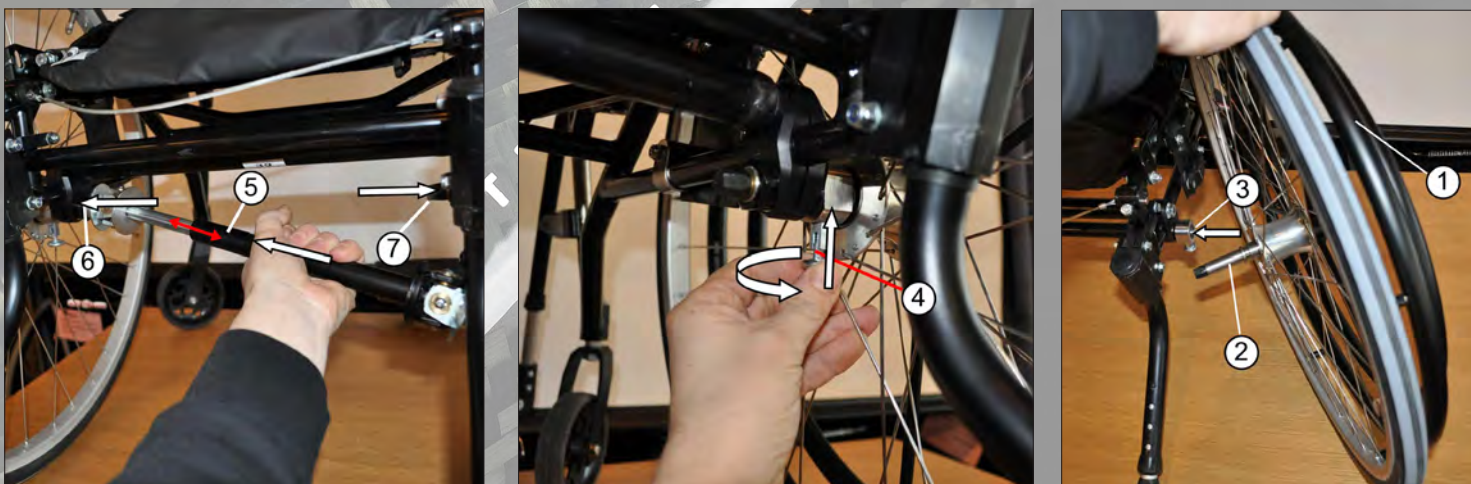

Loosen the driveshaft (5) by att compressing it and release it from the wheels..  
Release the locking pin (4) by pulling it downwards and then turn it a quarter turn to release the rearwheels.  
Pull out the wheels(1) from the attachments.(3)

**NOTE! This must not be done with the user in the chair!**