

Instructions for use Push handle H/A

Adjustment and use of the push handles

1. Active position. Release the quick lock. Twist the push handle into the correct position and pull it out to its full extent. Tighten the quick lock to lock the push handle into active position (figure 1)
2. Transport position. The push handles can be lowered and twisted so as not to protrude. Do this by loosening the quick lock and pushing the handle down. Tighten the quick lock to lock the handle in transport position (figure 2).

NB! Always make sure that the 'button' is 'out'. It acts as a safety stop.

A tip:

If you wish, you can remove the handles (just allowing the mounting to remain in place) when you are not using them. You can easily do this by loosening the quick lock and pulling up the handle to full extent. Now push in the 'button' and pull the handle upwards. Don't forget to tighten the quick lock a little so that it doesn't fall off.



Figure 1



Figure 2