

FOOTREST S3 SWING

Troubleshooting for:
Footrest S3 Swing
art.no. 43911XX, 43912XX, 43913XX

Tools needed:
Allen key 3 mm
Allen key 6 mm
Knife



LINK TO
INSTRUKTION VIDEO



1) If the mechanism for loosening the footrest has jammed, even though you press the release button properly, you can follow this instruction.



2) Loosen this screw with allen key 3 mm.




3) Then use the allen key to push out the locking piston to release the footrest from the wheelchair.



4) Remove the footrest from the wheelchair.



5) Remount the screw to the footrest.

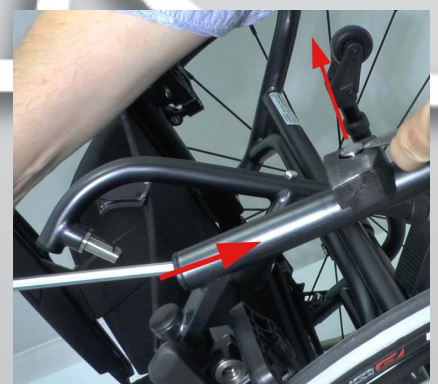

6) Use a suitable tool, e.g. a long allen key to push down the release mechanism in the frame to make the ejector to come out. Pull it out with e.g. a knife

panthera®



7) Take out the ejector.



8) Then tighten the attachment towards the frame properly with allen key 6 mm..



9) Remount the ejector.



10) Finally check the function of the locking mechanism of the footrest.



LINK TO
INSTRUKTION VIDEO