

## Instructions for assembly Low brake

### Assembly:

1. Check that the width of the brake is the same as the width of your chair.



Figure 2

3. Attach the brake section to the chassis. Tighten carefully.



Figure 4

5. Release the brakes, push the brake section back 3-4mm. First tighten the left brake and then the right brake. Make sure that the brakes are released inwards correctly when the brakes are unlocked. If this feels stiff you should twist the right hand brake up or down until you find the position where the brakes return easily when released.



Figure 1

2. Loosen the nut securing the clamps.



Figure 3

4. Pull to put the brakes on so that they are positioned against the tyres.



Figure 5

**Please turn over!**

# panthera

Panthera Production AB  
Gunnebogatan 26 , 163 53 Spånga , Sweden  
Tel: +46 (8) 761 50 40 Fax: +46 (8) 621 08 10  
panthera@panthera.se www.panthera.se

## Instructions for assembly Low brake

6. The distance (A) should be the same on both sides. On 24" standard wheels the distance (A) is around 230-234mm depending on how hard you want the brakes to lock. If you have 26" wheels the distance is around 255-257mm. Note that this measurement can vary depending on what type of tyre you have on your wheelchair, the patterns are different on different types of tyre.

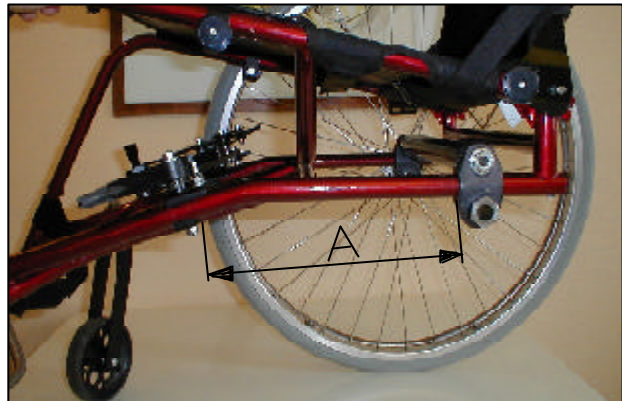


Figure 6



Figure 7

7. Remember that the brakes need to be cleaned and lubricated regularly - especially during the winter.