

Instructions for assembly Anti tip device Bambino

How to assemble your Bambino anti tip device:

1. Start by screwing on the 'Tip stop bushing' (1) using the screw (2), the two washers (3) and the nut (4).
2. Hook the lower spring (5), as shown in figure 1, in the 'spiral tension pin' (6).
3. Hook the same spring onto the screw (2). You need a long tool for this. Use a spoke or something similar to reach down through the back chassis frame (from above) and hook up the spring onto the spoke, tightening the spring on the screw.

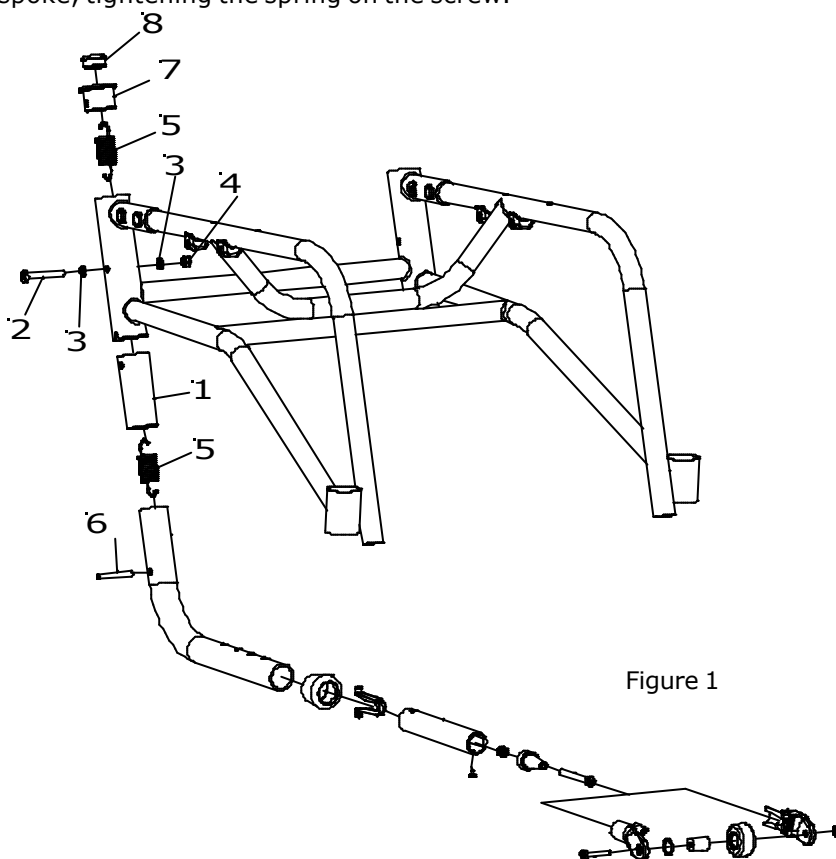


Figure 1

4. Now attach the upper spring (as shown in figure 1) around the screw (2).
5. Take a new spoke or similar tool, and reach down through the 'Spring fastener' (7). Hook the spring onto the spoke and tighten the spring around the tension pin in the spring fastener.
6. Check that both springs are securely in place by pulling the anti tip device, folding it in and out.
7. When everything is securely in place, insert the 'End plug' (8).