

SIDE GUARDS BAMBINO 3



Assembly instructions for:
Side guards Bambino 3
art.nr. 4235001, 4235002, 4235010

Tools needed:
Allen key 3 mm
Allen key 4 mm



1) Remove the rear wheels and place the wheelchair upside down. Take one bracket, left or right.



2) Put the bracket on the chassis as shown in the picture and close the bracket. See exploded view on page 3.



3) Fit screw M5x30 mm through the threaded bushing, welded on the chassis. Tighten with Allen key 3 mm.



4) Place the threaded bushing on it opposite side.



5) Tighten securely with a 4 mm Allen key.



6) Take screw M5x16 mm. Put it in the other hole and tighten firmly with a 4 mm Allen key.



7) Turn the wheelchair over, into upright position.



8) Then mount the side guard in the bracket. Repeat the procedure for the other side.

Adjust the side guard for 20, 22 or 24 " wheels

The side guard can be adjusted to fit different sizes of wheels. Parts 2 and 3 (see figure ??) are detachable to adjust the height of the sideguard to fit desired wheel size.

1. Use only part (1) for 20 " wheels.
2. Use part (1 and 2) for 22 "wheels.
3. Use part (1 and 2 and 3) for 24 " wheels.

