

# panthera®

## Assembly instruction Panthera X brake



The new X brake is designed to be mounted on Panthera X in the brackets, seen in **image 1**. The brackets are located on the underside of the frame tubes at the front, on both sides of the chassis.

On the brake you find two T-slot bolts that fits in the brackets on the chassis, see **image 2**.

When the T-slot bolts are mounted in the brackets you can adjust the brake along the T-slot. See **image 3**. Tighten the screws slightly and try to activate the brake.

To brake, you pull the handle sidelong and then pull it against the wheel, as **image 5** shows. The handle could be mounted on either the left or the right side.

When the correct adjustment of the brake is achieved the brake should cut approx. 5 mm into the tyres when activated, see **image 4**. The tyres should be fully inflated, approx 8 bar.

When the adjustments is done, tighten the screws slightly harder, **image 3**.


Done !



1



2

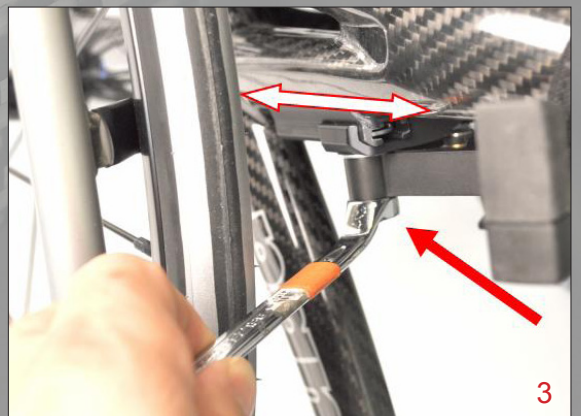

3



5



4