

BRAKE S3



Assembly instruction for:
Brake S3
also valid for brake lever extender
art.no. 4630000, 4630001, 4630002, 4630010
4630011, 4630012



[Link to video](#)

Tools needed:

Allen key 5 mm



1) The attachment for the brake should be mounted around the chassis tube.



2) The brake placed in an approx. positions. Adjusted later.




3) Tighten the screw slightly with allen key 5 mm.



4) Engage the brake and check that the brake blade cuts into the pumped tire approx. 5 mm.



5) If needed adjust the brake in depth to get the correct brake force.

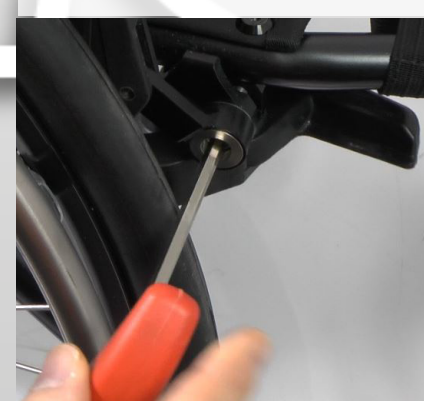

6) When the brake is properly adjusted tighten the screw firmly with allen key 5 mm.
Repeat the procedure 1-6 on the other brake.