

## Instructions for assembly Folding footrests

### Assembly:

1. Check to make sure you attach the correct plate to each side (right/left). On some footrests there is one for the right hand side and another for the left hand side. The difference can be seen by checking the angles as if a mirror were placed between them.

2. Place the end plugs (5) on the ends of the tubing of the frame where the footrest is normally attached.

3. Then push the tubing of the footrest (1) into the frame. There are 5 possible holes to choose from for different height adjustments. Make sure that you choose the same height for the right and left hand sides.

4. Tighten firmly using the same screw attaching the footrest.

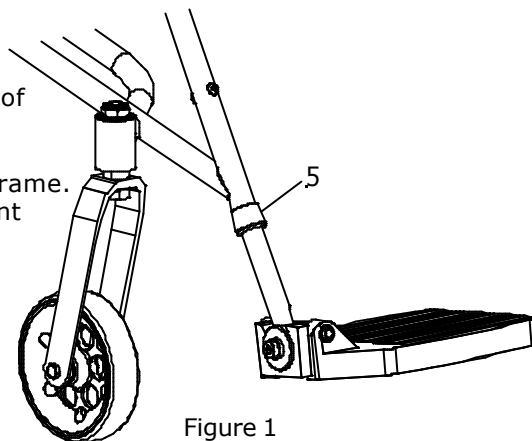


Figure 1

### Adjusting the angle of the plate:

1. You do this by first unscrewing the stop screw (3) on the side of the joint block (2) with Allen key NV2.5. This screw should be loosened and unscrewed so that the ends are even with the surface of the joint block. When you receive the plate from us this screw will not have been tightened the first time.

2. After loosening the stop screw, loosen the centre bolt of the joint block (4) using Allen key NV5 (this key is included with the original tools for the wheelchair). You will then be able to turn the plate to the desired angle.

Tighten the following:

1. First, the centre bolt of the joint block. Tighten this hard.

2. Screw in the stop screw until you feel some resistance. Continue to screw this 1-1.5 revolutions. The stop screw will then sink into the plastic and will ensure that the plate cannot be turned to alter the angle.

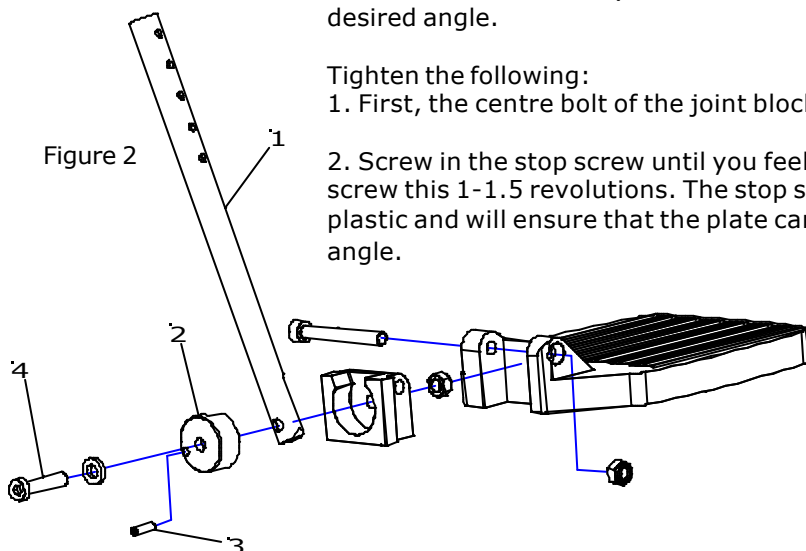


Figure 2

**NB!** Stop screw no.3 is not screwed in when you receive the plate from us.  
(See 'Adjusting the angle of the plate' for instructions).