

Instructions for assembly Raising kit 26"

How to assemble your raising kit 26":

1. Start by dismantling the caster fork as shown in figure 1. Remove the end plug (2) and loosen the nut (6). Remove the washer (5), and tap out the fork (1). Take care not to damage the bushings (3 and 4) attached to the chassis.

2. Tap out the bushings (3 and 4) carefully.

3. Take the fork and unscrew the fork pin as shown in figure 2.

4. Replace your present fork pin with the longer fork pin for the height extension. Screw this on. See position 1 in figure 3.

5. You now assemble the castor fork with the new fork pin on your chair. Start by lubricating the bushing casing (7) in figure 3 using locking oil (such as Loctite). See figure 3.

6. Insert the bushing (3) into the casing (7) and then push the casing into the chassis as shown in figure 3.

7. Rub some locking oil around the bushing (4) before pushing it into the chassis.

8. Now push the castor fork up through the casing, place the washer on the upper bushing and tighten the nut securely. Insert the end plug. It should now look like figure 4.

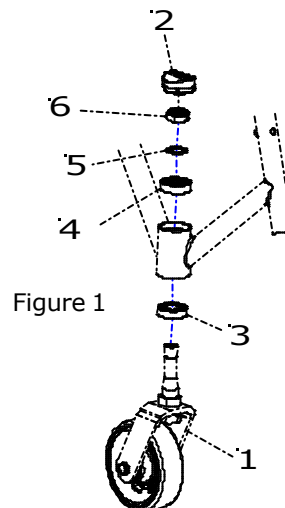


Figure 1

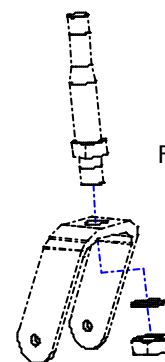


Figure 2

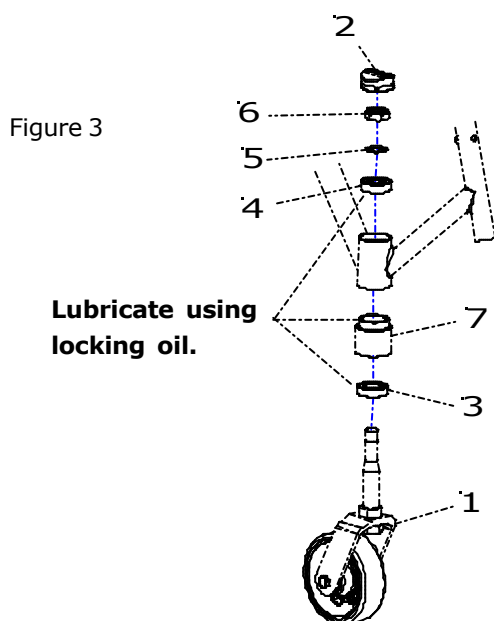


Figure 3

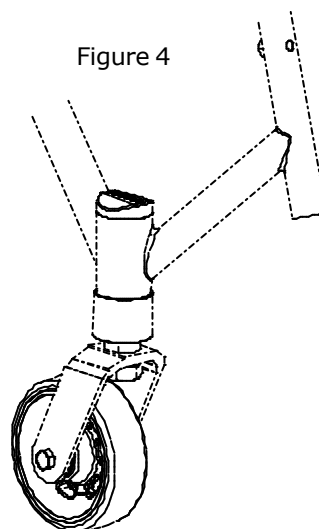


Figure 4