

Instructions for assembly Footrest Micro

How to assemble the micro footrest:

1. Place the 'mounting bracket for the footrest' (2) against the footrest (1) and add washers (3) and bolts (4 and 5) as shown in figure 1.

2. Push the bolts through the anchor holes in the chassis of the chair.

3. Add washers (3 and 6) to the bolts and tighten the nuts (7) securely.

4. Repeat this procedure on the other side.

NB! You can adjust the height of the foot plate by mounting it at different positions on the mounting bracket (2).

