

Instructions for assembly Push handles H/A

1. Start with the chair. Loosen the back upholstery and remove the end plugs on the upper part of the backrest frame (figure 1).

2. Place the chair on its side. Insert the locking block into the tubing of the backrest frame (where the end plugs were to start with) so that the holes in the backrest frame and those in the locking block are aligned.

3. Then take the push handle mounting and the short bolt with a washer. Insert the bolt into the mounting and screw the bolt into the locking block positioned on the backrest frame (figure 2). Tighten the bolt slightly. You can now replace the end plug in the backrest frame.

4. Place the hinge clamp round the backrest frame bar and insert the long screw with the washer through the clamp and the push handle mounting (figure 3). Tighten slightly.

5. Assemble the push handle in the mounting by pushing in the 'button' on the lower part of the handle and inserting it in the mounting (figure 4). Adjust using the handle so that the mounting is straight.

6. Now tighten the first bolt securely (figure 2) and then the second bolt (figure 3).

7. Tighten the quick lock so that the handle is securely attached.



Figure 1



Figure 2



Figure 3



Figure 4