

Assembly instructions for the heel band

You will need the following parts to assemble a heel band:

- 1 heel band right/left
- 1 pin
- 1 screw
- 1 self-adhesive Velcro band

How to assemble the heel band onto the footplate:

1. Start by drilling a hole with a diameter of 5 mm in the round notch on the underside of the footplate, see figure 1.
2. Place the pin into the heel band. Hold the pin above the hole on the upper side of the footplate and screw the screw into the hole from the underside of the footplate, see figure 2.
3. Find the height most suitable for the heel band on the chassis frame, see figure 3.
4. Loosen the heel band.
5. Assemble the self-adhesive Velcro onto the chassis frame where the heel band will be attached.
6. Attach the heel band. Ready!



Fig. 1

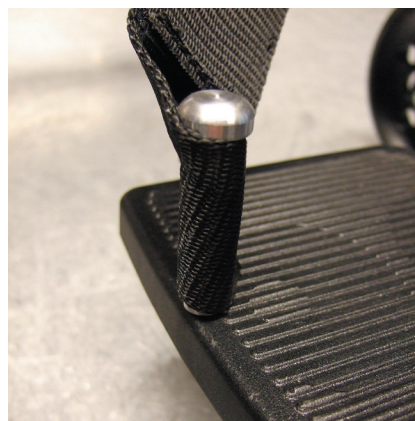


Fig. 2

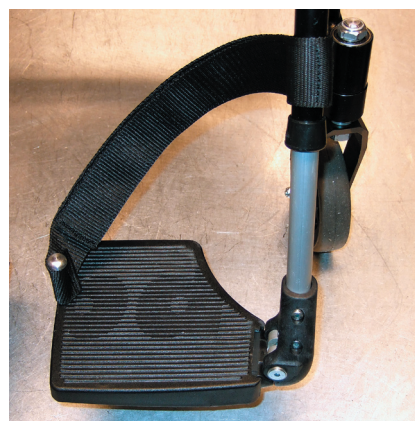


Fig. 3