

## Fault tracing routine Swing

### If the valves fail to lock:

Clean the locking mechanism according to the cleaning instructions 882 11 73 (available on our website [www.panthera.se](http://www.panthera.se)).

### If the leg support jams:

Use a small screwdriver to push out the valve inside the locking mechanism, see figure 1.

### If the leg support button fails to function:

1. Remove the leg support from the chassis (see above if it is jammed).
2. Loosen the plastic plug at the top of the leg support, see figure 2. If you are unable to loosen the plastic plug with your fingers you can carefully use a screwdriver.
3. Push the spring pin down in the tube with a small screwdriver. Keep the spring pin out by holding onto the protruding pin, see figure 3. Remove the screwdriver.
4. Align the front section of the button in the track, see figure 4.
5. Check to make sure the button functions. The pin should protrude when the button is pushed in, see figure 5.
6. Replace the plastic plug and assemble the leg support onto the chair. Ready!  
Contact Panthera for advice if these instructions do not help.



Fig. 1

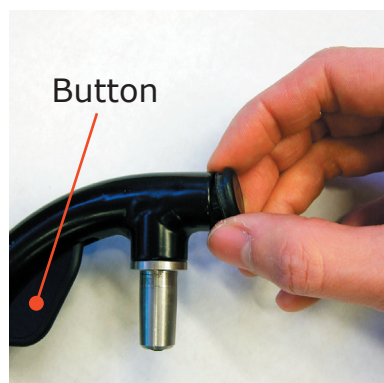


Fig. 2

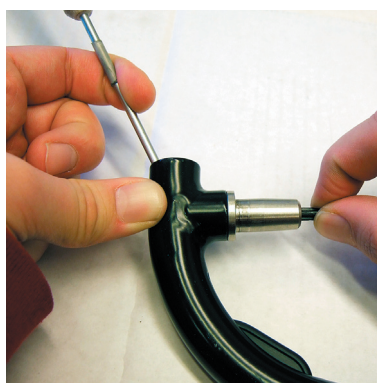


Fig. 3

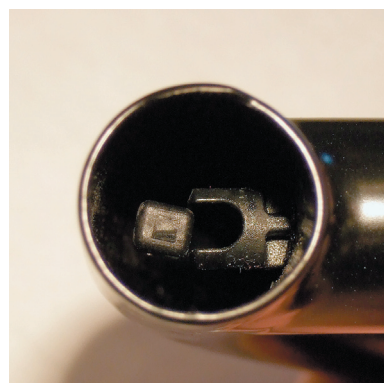


Fig. 4



Fig. 5

# panthera

Panthera Production AB  
Gunnebogatan 26, SE-163 53 Spånga Sweden  
Office: + 46 8 761 50 40 Fax: + 46 8 621 08 10  
[panthera@panthera.se](mailto:panthera@panthera.se) [www.panthera.se](http://www.panthera.se)