

GELPLATTA

Monteringsanvisning för: Gelplatta på armstöd

art.nr. 4401251 Gelplatta självhäft.190*50*10mm par




QR Länk till video

Verktyg som behövs:


T-sprit el. liknande



1) Använd T-sprit eller liknande för att rengöra ytan som gelplattan ska fästas på.

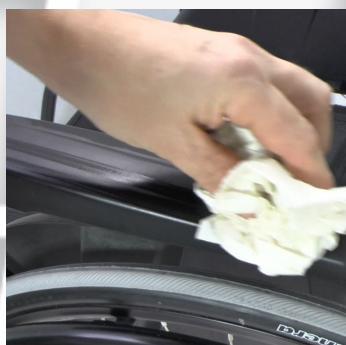

2) Rengör ytan ordentligt från fett och smuts så att den självhäftande gelplattan kan fästa mot ytan.

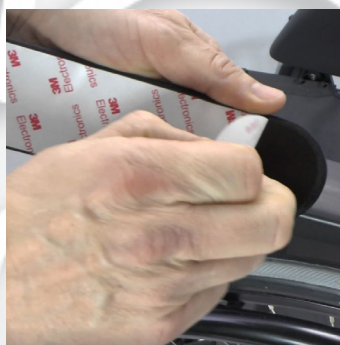


3) Avlägna skyddspappret från gelplattans undersida.



4) Placera gelplattan på armstödet.



6) Tryck fast den ordentligt.

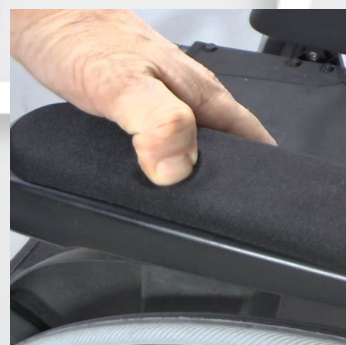

7) Nu har armstödet fått en mjuk och behaglig yta som skonar armbågen..



GEL PAD

Assembly instruction for: Gel pad on arm support

art.no. 4401251 Gel pad stick-on 190*50*10mm pair

Tools needed:
Methylated spirits



QR link to video



1) Use methylated spirits or similar to clean the surface where you will attach the gel pad.



2) Clean the surface thoroughly from grease and dirt.



3) Remove the protection paper from the underside of the gel pad.



4) Place the gel pad on the arm support.



6) Press hard on the gel pad.



7) Now the arm support has a soft and comfortable surface that is nice to your elbow.