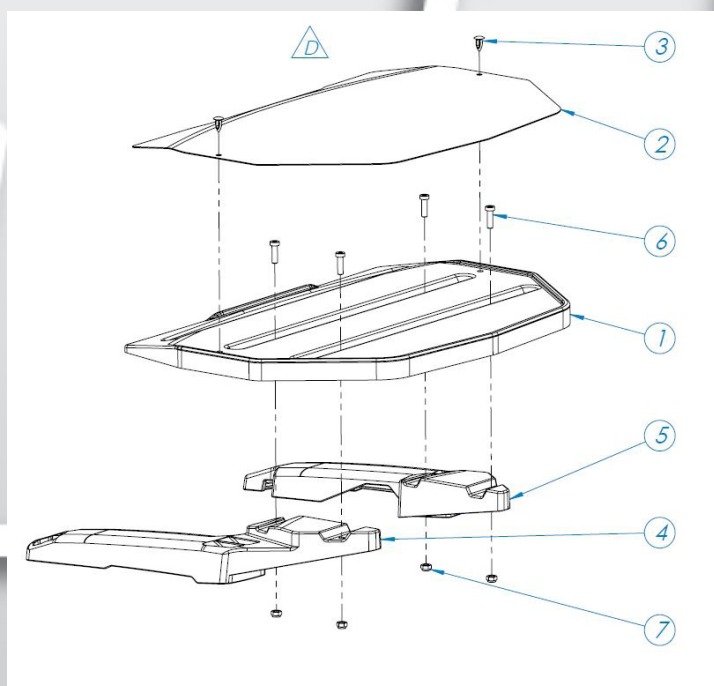
Monteringsanvisning för:
Bord S3


art.nr. 6810020 Bord S3

Max. belastning 10 kg

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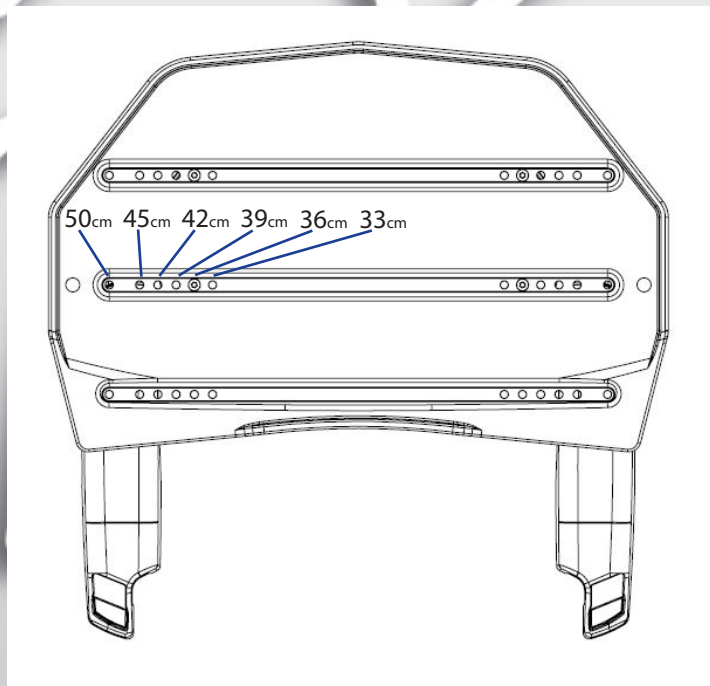1. Sprängskiss Bord S3

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

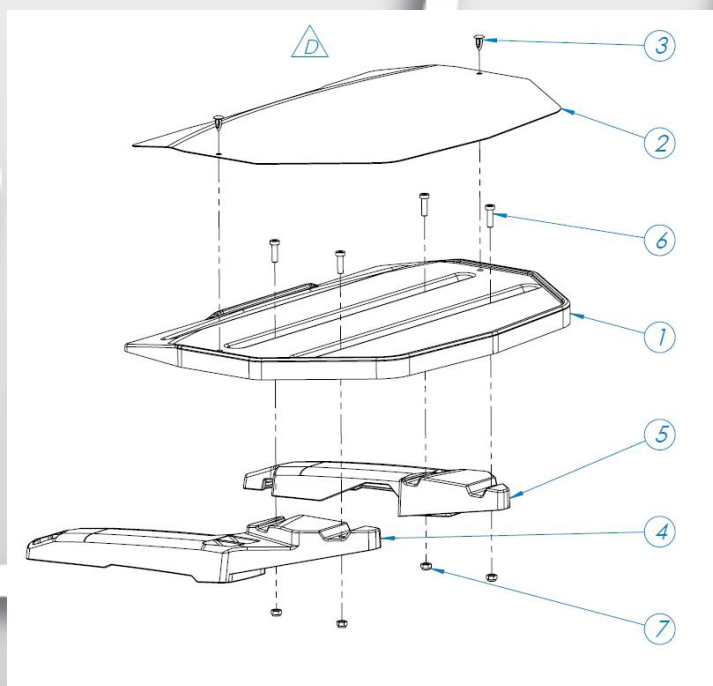
Tools needed:
Allen key 4 mm

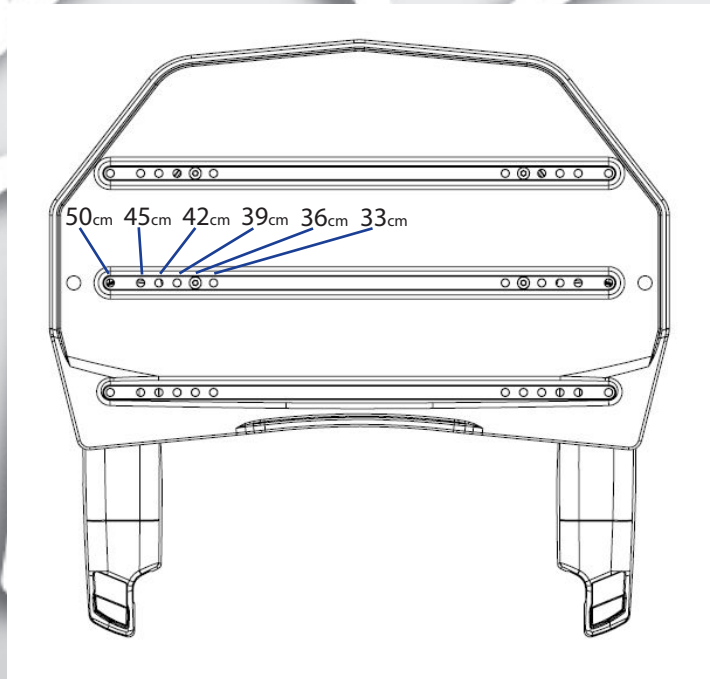1. Exploded view Table S3

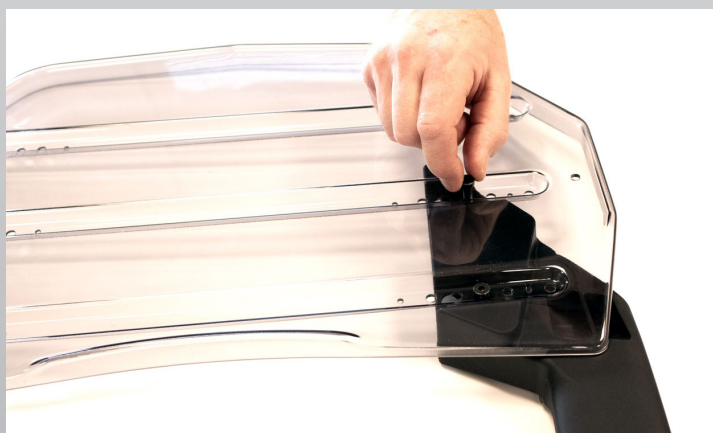2. Choose desired width and depth position.

3. Place one of the tables attachments (4 and 5) in desired position. Place bolts (6) from tables (1) upper side.

4. Mount the lock nuts (7) from underneath. Tighten the bolts. Repeat same steps on the other side.

5. Remove the protective film from the loose cover top (2). The cover top is placed on the table and fastened with the supplied black locking plugs (3) in the corresponding holes on the table top.

Use of the table

6. To mount the table on the armrest, select the desired depth position on the armrest. Angle the table upwards, thread on the mounting sides on the back edge of the armrest and place the table on the armrests.

To remove the table, angle the front edge of the table upwards and press the table slightly backwards and downwards to slide it off the armrests.

Using the table on wheelchairs with 0° or 4° seat angle.

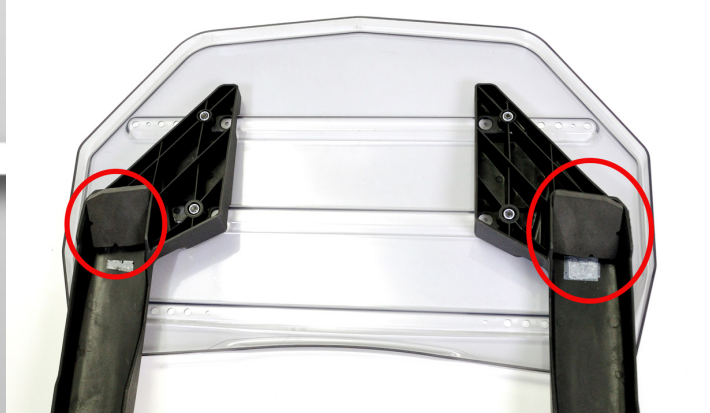
7. To use the table on wheelchairs with 4° or 0° seat angle, that is, S3 Swing, S3 Swing Low 0° and S3 0°, the table is delivered with 4 p. self adhesive angel adapter.

2 p. 10 mm, R and L

2 p. 18 mm, R and L.

They make the table surface flat even with these seat angles.

*Use 2 p 10 mm Angel adapter for 4° wheelchairs
Use 2 p 18 mm Angel adapter for 0° wheelchairs*

8. Place the **table upside down** and remove the protective plastic from the angel adapter. Note that it is a left and a right version.

Turn them so the two V-shaped slots on the angel adapter is pointing out wards-down wards, as in the image, and stick the angel adapter to the front of the table attachment when the table is upside down.

Then mount the table as in fig 6.